

INSTRUCTION FOR USE OF PERSONAL PROTECTIVE EQUIPMENT

MERCATOR yellow

The instruction below should be used in conjunction with detailed information on the packaging.

Description of the product

Protective gloves (for minimal risk only), latex, powder-free, non-sterile
 Colours : yellow
 Sizes : S, M, L, XL
 Quantity in packaging : 240 x 1 pair
 Shelf life : 5 years from manufacturing date

Storage instructions

Do not expose to direct sunlight, ozone sources or sources of fire. Store in a dry and cool place, at a temperature of 5-35°C. Do not keep in direct vicinity of solvents, oils, fuels and lubricants.

Intended use

These are protective gloves, latex, powder-free, non-sterile. Gloves are classified as a Personal Protective Equipment Category I, for minimal risk only. Gloves designed to protect against minimal risks, i.e. prolonged contact with water. Their design and labelling corresponds to the requirements of the European Regulation 2016/425 on Personal Protective Equipment. Gloves should be used solely according to their intended application.

Manufacturer

MERCATOR MEDICAL S.A.
 ul. H. Modrzejewskiej 30
 31-327 Kraków, Polska

Declaration of Conformity and this instruction for use available under below web address:
<https://mercatormedical.eu>

Precautions and indications for use

Dry hands before taking the gloves out from the packaging. Before usage, inspect the gloves for any defect or imperfections. If the gloves get punctured, torn or broken during their use, take them off and put on the new ones. Avoid using gloves dirty in the inside as they may cause irritation leading to skin inflammation or more serious damages.

It is recommended to check that the gloves are suitable for the intended use because the conditions at the workplace may differ from the type test depending on the temperature, abrasion and degradation. The gloves should not be used in contact with open fire and to protect against any sharp tools. The gloves are not intended for welding, electric shock protection, ionizing radiation or from the effect of hot or cold objects.
















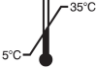

Components / hazardous components

Product contains natural rubber latex which may cause allergic reactions including anaphylactic responses. Components used in making gloves may cause allergic reactions in some people. Some gloves may contain components known to be a possible cause of allergy for person allergic to them, who may develop contact irritation and/or allergic reaction. In case of an allergic reaction consult a doctor.

Disposal

Dispose of in accordance with applicable regulations.

Symbols used on the packaging

	Personal Protective Equipment		Expiry date		Model number		Package can be treated as municipal waste
	Manufacturer		Date of manufacture		Temperature limitation		Latex gloves
	Lot / batch number		Keep dry		Product quality is not ensured if the package is damaged		Non-sterile
	Catalogue number		Keep away from sunlight		Recyclable packaging		Consult instructions for use
	Indicates compliance with the requirements of Ukrainian market						

MERCATOR yellow MODEL #1537

REF NO: RG10004002-05_1537

PPE classification & compliance

Gloves are category I, for minimal risk only Personal Protective Equipment as per Annex I of the Regulation 2016/425 and comply to standards:

EN ISO 21420:2020, EN ISO 374-2:2019.

